

# セティルザイム フォーム 2

酵素系洗浄剤(泡タイプ)

セティルザイムフォーム2は2種類の酵素を配合した治療・外科用器具向けの泡タイプの酵素系洗浄剤です。使用直後の器具を泡で覆うことにより、乾燥による汚れの固着を防ぐだけでなく、血液などのタンパク質やデンプン質の汚れを溶解除去し、最終洗浄を容易にします。インプラントなどの手術時の器具類にお使いください。

## 使用方法

シールをはがし、そのままスプレーを取り付けて使用してください。

- ①15~20cm離して器具に直接スプレーし、全体を泡で覆ってください。
- ②泡で覆われた状態で3分程度放置してください(汚れの程度によって適宜延長してください)。
- ③水洗し、泡を十分に洗い流してから、滅菌等必要な処置を行ってください。

## 注意

- ・作業時は必ずゴム手袋及び保護メガネを着用すること。
- ・目に入った場合には、すぐに大量の水で洗浄し、眼科医の診断を受けさせること。
- ・万一飲み込んだ場合には、多量の水又は牛乳を飲ませ、無理に吐かせないで医師の診断を受けさせること。
- ・塗装面等は目立たない部分でシミ、変色がないことを確認してから使用すること。
- ・30°Cを超えない場所で保管すること。
- ・少し強めにトリガーを引くと液だれせず、キレイに泡立ちます。

組成 プロテアーゼ・アミラーゼ・界面活性剤  
容量 24 oz.(709mL)

無リン 中性

輸入発売元

 **ネオ製薬工業株式会社**  
〒150-0012 東京都渋谷区広尾3丁目1番3号  
Tel (03) 3400-3768(代) Fax (03) 3499-0613

製造元

**CETYLITE INDUSTRIES, INC.**

XI

# Cetyl-Zyme

## Protease-Amylase Dual Enzymatic Detergent

# Foam 2

## AFTER

### Implant Surgery Endodontic Surgery Periodontal Surgery

### Bacteriostatic • Non-Corrosive • Neutral pH

Cetyl-Zyme Foam keeps instruments moist and begins breaking down blood and tissue proteins and starches until instruments can be thoroughly cleaned.

NET CONTENTS:  
24 oz.(709ml.)



Manufactured by

**CETYLITE INDUSTRIES, INC.**

**PIONEER LEADER VISIONARY**

9051 River Road  
Pennsauken, NJ 08110-3293

Made in USA  
Rev.4/08